


SAFETY LANCET

INSTRUCTION FOR USE

INDICATION

The sterile safety lancets for single use are mainly applied for collecting blood samples for hospital clinical tests. The precise safety lancets guarantee by simple and comfortable handling a very gentle application and protects patient and user against needle injuries.

INSTRUCTION FOR USE

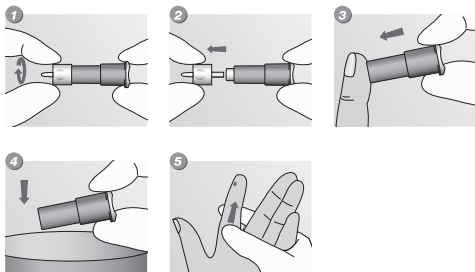

1. Rotate the twisting cap less than half a round
2. Pull out the twisting cap
3. Place the device on the puncture site and push to start
4. Discard lancet into a sharps container.
5. Press lightly on the finger towards the puncture site to obtain adequate blood sample

SAFETY LANCET

INSTRUCTION FOR USE


CAUTIONS

- The product is sterile for single use only.
- Do not use if package is open or damaged.
- Do not use in case of loose or detached sterile cap.
- Discard the product after single use.

PRECAUTIONS

- The product is sterile and pyrogen-free.
- The product is sterilized by gamma radiation.
- Store the product in a cool and dry place. Keep away from direct sun, heat and humidity.
- Read the instruction for use before application.

FOR SINGLE USE ONLY

 The product may be used in sterile condition, one-time with one single patient. Cleaning or re-sterilization are excluded. Re-using single-use products bears potential risks for both patients and users. Impairment of material properties and inadequate sterility associated with such re-uses may lead to illness, damage or death of the patient.

CE
0197

STERILE R

**Promised**[®]
Promised Hangzhou Meditech CO.,LTD.
No.12 Longtan Road, Cangqian Street, Yuhang
District, Hangzhou City, Zhejiang, China
www.promisedmed.ca

EC REP

MT Promed Consulting GmbH
Altenhofstr. 80 66386 SL
Ingbert Germany.